

EUGENOL

Cloves Oil (Extra Pure)

TECHNICAL & SAFETY INSTRUCTIONS FOR USE



Intended Use: Eugenol is used as an obtundent for the relief of minor dental pain. Reacts with zinc oxide to form a sedative dental dressing or lining under a non- composite resin restoration.

Composition: Oil of Cloves 98% pure.

Areas of application: Insulating cavity lining allowing placement of amalgam restorations and other cements. Temporary seal for medicament dressings. Soothing temporary filling for sensitive dressings.

Procedure to use material: Place the required amount of Zinc oxide powder and Eugenol a short distance apart on a clean, smooth glass slab. Using a suitable spatula, progressively incorporate portions of powder into the liquid, and mixing thoroughly after each increment addition. Reach to cohesive consistency of paste before filling. Mixing time 1-1.5 min. according to size of mix, Place sufficient of the mix into the dried cavity and gently pat into position to prevent the instrument sticking during this placement, periodically dip the end of the instrument into the surplus dry powder on the mixing slab.

Storage: Keep Away from Direct Sunlight, Keep Away from Moisture, Always keep the Bottles and packets tightly closed once opened. Store at Temp. between 5°C to 30°C, Shelf life not affected after opening if stored in a cool dry place, If not to be used for a longer duration, please keep them in refrigerator, Allow material to reach room temperature prior to use

Disposal Conditions: To be disposed off based on the prevailing rules of the country in which it is used.

Warning: Avoid prolonged or repeated contact with skin (allergic contact dermatitis), and eyes. Avoid prolonged inhalation. Do not take internally. Eye contact if the product comes in contact with eyes it may cause irritation. In case of contact with eyes, rinse immediately with plenty of water and seek medical attention. Skin Contact if comes in contact with Skin, it may cause skin sensitization, If contact with skin occurs immediately wipe off thoroughly with cotton and alcohol and then wash well with soap and water. If skin rash and sensitization or other allergic reaction occurs discontinue use and seek medical attention. -It should not be used with patients who have a history of severe allergic reaction to any of the components.

Presentation : 1 x 15 ml Ref No. ED13-01, 1 x 100 ml Ref No. ED13-02, Glass Bottle.

Shelf Life: 3 years from date of manufacturing.

NOTE: TO BE USED BY DENTAL PROFESSIONALS ONLY.

Manufactured By:
Golden Bird International
Plot No. A-4 Vishwakarma
Industrial Estate, Village Panchli
Khurd Meerut-250002 India, info@indiagbi.com
www.dentakris.com, GOLDEN-Meeru-UP/M/MD/005962
®-Registered Trademark Only

Symbol for "BATCH CODE" Symbol for "USE BY" Symbol for "NON STERILE" Symbol for "KEEP DRY" Symbol for "CAUTION"
 SYMBOL FOR EUROPEAN COMMUNITY AUTHORIZED REPRESENTATIVE Symbol for "KEEP AWAY FROM SUNLIGHT"
 Symbol for "KEEP OUT OF THE REACH OF CHILDREN" Symbol for "MANUFACTURER NAME & ADDRESS" Symbol for "CONSULT INSTRUCTION FOR USE" Symbol for "CATALOGUE NUMBER" Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"
 Symbol for "TEMPERATURE LIMITATION"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding.