

G.P CLEAN

Softening Gutta- Percha

TECHNICAL & SAFETY INSTRUCTIONS FOR USE



INTENDED USE: Gutta-Percha lemon oil based solvent is used for removing gutta percha & temporary cement at the time of root canal treatment.

COMPOSITION: Lemon oil, organic solvent, etc.

DESCRIPTION: Gutta-Percha is recommended for softening and removing gutta-percha. It is especially recommended during 1. For removing of gutta-percha in order of retreatment of the root canal, in case of more difficult removal of Gutta percha from curved canals and highly condensed and hardened gutta-percha and sealer, Filling the 2. Root canals using a single pin or with the method of side condensation of gutta-percha. **3. DIRECTIONS FOR USE: REMOVING GUTTA-PERCHA IN ORDER OF RETREATMENT OF THE ROOT CANAL:** 4. Insert 1-2 drops of the preparation into the tooth cavity and cause the enetration of the solvent using a File i.e. K-file size 25. 5. Remove the softened gutta-percha after approximately 2 min. with a manual canal tool i.e. K-file size 20 and 15 or H-file 6. If necessary apply the next dose of the product, placing it inside the partially empties canal 7. Rinse the canal well with sodium hypochlorite using at least 2 ml apply the saliva Ejector. 8. Stop applying the solvent when the file gets into 1/3 of the canal apical depth. 6. Remove the material from the canal and rinse the canal until the rinsing liquid is clean.

CONTRAINDICATIONS: Do not use the product in the case of hypersensitivity to orange oil. Do not use the product for removing gutta-percha from the apical zone. Do not apply the material beyond the apical foramina of tooth. It is recommended to use cofferdam during application of the product. Use protective measures (glasses, gloves) when working with the product. Avoid prolonge inhalation. Do not take internally Contact with the skin may cause irritation, burns of hypersensitivity. If contact with skin occurs immediately wipe off thoroughly with cotton and alcohol and then wash well with soap and water. If skin rash and sensitization or other allergic reaction occurs discontinue use and seek medical attention. If the product comes into contact with the eyes, rinse thoroughly with fresh water and consult a physician.

Warning: Avoid prolonged or repeated contact with skin (allergic contact dermatitis), and eyes. Avoid prolonged inhalation. Do not take it internally. Contact with the skin may cause irritation, burns of hypersensitivity. If contact with skin occurs immediately wipe off thoroughly with cotton and alcohol and then wash well with soap and water. If skin rash and sensitization or other allergic reaction occurs discontinue use and seek medical attention. If the product comes into contact with the eyes, rinse thoroughly with fresh water and consult a physician. It should not be used with patients who have a history of severe allergic reaction to any of the components.

PRECAUTIONS FOR USE: Always replace cap immediately after use, keep jar in a dry place below 25° C.

PRESENTATION : 15 ml. REF NO.ED04-01, Glass bottle.

STORAGE CONDITIONS: The product should be stored below 25° C.

DISPOSAL CONDITIONS: To be disposed off based on the prevailing rules of the country in which it is used.

EXPIRY: 3 years from the date of manufacturing.

NOTE: TO BE USED BY DENTAL PROFESSIONALS ONLY

Manufactured By:
Golden Bird International
Plot No. A-4 Vishwakarma
Industrial Estate, Village Panchli
Khurd Meerut-250002 India, info@indiagbi.com
www.dentakris.com, GOLDEN-Meeru-UP/M/MDI/005962
®-Registered Trademark Only

LOT Symbol for "BATCH CODE" **☼** Symbol for "USE BY" **☼** Symbol for "NON STERILE" **☼** Symbol for "KEEP DRY" **☼** Symbol for "CAUTION"
EC/REP SYMBOL FOR EUROPEAN COMMUNITY AUTHORIZED REPRESENTATIVE **☼** Symbol for "KEEP AWAY FROM SUNLIGHT"
☼ Symbol for "KEEP OUT OF THE REACH OF CHILDREN" **☼** Symbol for "MANUFACTURER NAME & ADDRESS" **☼** Symbol for "CONSULT INSTRUCTION FOR USE" **REF** Symbol for "CATALOGUE NUMBER" **☼** Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"
☼ Ssymbol for "TEMPERATURE LIMITATION"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding.