

HYPO CLEAN

Root Canal Disinfectant (Sodium Hypochlorite)

TECHNICAL & SAFETY INSTRUCTIONS FOR USE



Indications : During mechanical canal widening it removes the non-vital pulp debris. It cleans the canal, removes the smear layer in order to expose the dentin root canal orifices. It prevents from the tooth discoloration that might occur after filling the non-rinsed root canal.

Composition : Sodium Hypochlorite, Purified water.

Instructions for use : Insert the liquid Hypo clean 3% / Hypo clean 5% into the root canal by means of the syringe with needle with side port. Rinse the canal copiously, approximately 10 ml for one canal. Sodium hypochlorite contained in the product inactivates immediately in contact with organic substance. While concentration of sodium hypochlorite is decreasing, time of rinsing and volume should be necessarily simultaneously increased.

Cautions : During root canal preparation do not use too heavy stream of the liquid because it can cause the liquid penetration to the periapical tissues and damage them, characterized by pain and swelling of soft tissues.

Contraindications : Do not use product Hypo clean 3% / Hypo clean 5% for patients with hypersensitivity to the product's components.

Storage Conditions : Store at temperature between 10°C - 24°C. Close the cap carefully after use. Keep away from moisture. Shelf life not affected after opening if stored in a cool dry place.

Warning : Hypo clean 3% / Hypo clean 5% can irritate soft tissues of the oral cavity, eyes and skin. Necessary use of rubber-dam while working with the product. Recommended use of personal protective equipment (glasses, protective visor, gloves). Avoid contact with eyes, soft tissues of oral cavity and skin. In case of accidental contact rinse with plenty of water.

Disposal conditions : To be disposed off based on the prevailing rules of the country in which it is used.

Presentation :

Ref: ED01-01

3%, 500 ml Plastic Bottle.

Ref: ED02-02

5%, 500 ml Plastic Bottle.

Shelf Life : 3 years from the date of manufacture.

NOTE: TO BE USED BY DENTAL PROFESSIONALS ONLY.

Manufactured By:
Golden Bird International
Plot No. A-4 Vishwakarma
Industrial Estate, Village Panchli
Khurd Meerut-250002 India, info@indiagri.com
www.dentakris.com, GOLDEN-Meeru-UP/MMD/005962
®-Registered Trademark Only

Symbol for "BATCH CODE" Symbol for "USE BY" Symbol for "NON STERILE" Symbol for "KEEP DRY" Symbol for "CAUTION"
 SYMBOL FOR EUROPEAN COMMUNITY AUTHORIZED REPRESENTATIVE Symbol for "KEEP AWAY FROM SUNLIGHT"
 Symbol for "KEEP OUT OF THE REACH OF CHILDREN" Symbol for "MANUFACTURER NAME & ADDRESS" Symbol for "CONSULT"
INSTRUCTION FOR USE Symbol for "CATALOGUE NUMBER" Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"
 Symbol for "TEMPERATURE LIMITATION"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding.