

IODOCAL

Clacium Hydroxide Iodoform Paste

TECHNICAL & SAFETY INSTRUCTIONS FOR USE



INTENDED USE: Iodocal Paste is a premixed calcium hydroxide, Barium Sulphate and Iodoform paste for root canal treatment and is used for following applications: 1. Disinfects canals. 2.

Inhibits bacterial growth. 3. Prevents flare-up. 4. Promotes per apical healing. 5. Treats infected root canals, abscesses and per apical lesions. 6. Treats traumatic injuries, root resorption, fractures and perforations. 7. Treats pus, haemorrhage, exudation and weeping canals. 8. Stimulates apexification and apex genesis.

COMPOSITION: Calcium Hydroxide Powder, Barium sulphate, Iodoform, etc

DESCRIPTION: Iodocal is a strong antibacterial dressing for routine use between office visits to disinfect root canals and for complicated cases when extended calcium hydroxide therapy is indicated. Iodocal can be used alone as a temporary canal disinfectant, for vital and non-vital pulpotomy, and for pulpectomies in deciduous teeth.

DIRECTIONS FOR USE: Using good endodontic procedures, cleanse and prepare the root canal in the conventional manner. Dry the canal. Place a new disposable applicator tip securely on the luer lock syringe. Fill the canal by applying gentle pressure on the syringe plunger of Iodocal. Do not force the plunger. The paste can also be placed into the canal with a lentulo spiral. Remove excess at the orifice with a sterile cotton pellet. Seal access opening with your preferred cement. Discard applicator tip and cap syringe immediately after use. Iodocal is easily removed with file and irrigation with sodium hypochlorite.

WARNING: Direct pulp sealing in infected dentine; affected inflammatory pulp sealing; emporary root canal filling with purulent periodontitis involving exudation. The product is counter indicated for use on patients with known hypersensitivity to any of its components Avoid prolonged or repeated contact with skin (allergic contact dermatitis), and eyes. Avoid prolonged inhalation. Do not take internally The high pH value of Iodocal may cause burns of mucosa and skin. If contact with skin occurs immediately wipe off thoroughly with cotton and alcohol and then wash well with soap and water. If skin rash and sensitization or other allergic reaction occurs discontinue use and seek medical attention. If the product comes into contact with the eyes, rinse thoroughly with fresh water and consult a physician.

PRECAUTIONS FOR USE: Always replace cap immediately after use, keep syringes in a dry place below 25° C.

STORAGE CONDITIONS: The product should be stored below 25°C.

DISPOSAL CONDITIONS: To be disposed off based on the prevailing rules of the country in which it is used.

Presentation : 1 x 1.5 gm RefNo. ED10-01, 1 x 3 gm RefNo.ED10-02, 2 x 1.5 gm RefNo. ED10-03, 2 x 3 gm RefNo. ED10-04 Applicator Tips.

EXPIRY: 3 years from the date of manufacturing.

NOTE: TO BE USED BY DENTAL PROFESSIONALS ONLY

Manufactured By:
Golden Bird International
Plot No. A-4 Vishwakarma
Industrial Estate, Village Panchli
Khurd Meerut-250002 India, info@indiagbi.com
www.dentakris.com, GOLDEN-Meeru-UP/M/MD/005962
®-Registered Trademark Only

LOT Symbol for "BATCH CODE" **U** Symbol for "USE BY" **NS** Symbol for "NON STERILE" **☞** Symbol for "KEEP DRY" **⚠** Symbol for "CAUTION"
EC/REP SYMBOL FOR EUROPEAN COMMUNITY AUTHORIZED REPRESENTATIVE **☼** Symbol for "KEEP AWAY FROM SUNLIGHT"
👶 Symbol for "KEEP OUT OF THE REACH OF CHILDREN" **🏠** Symbol for "MANUFACTURER NAME & ADDRESS" **📖** Symbol for "CONSULT INSTRUCTION FOR USE" **REF** Symbol for "CATALOGUE NUMBER" **🚫** Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"
🌡 Ssymbol for "TEMPERATURE LIMITATION"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding.