

IODOCAL

Calcium Hydroxide Iodoform Paste

TECHNICAL & SAFETY INSTRUCTIONS FOR USE



Intended Use: Iodocal Paste is a premixed calcium hydroxide–iodoform paste used for the following dental applications: Disinfection of root canals. Inhibition of bacterial growth. Prevention of flare-ups associated with periapical healing. Treatment of infected root canals, abscesses, and periapical lesions. Management of traumatic injuries, root resorption, fractures, and perforations. Apexification and apexogenesis procedures. Temporary root canal dressing between appointments.

Composition: Calcium Hydroxide Powder, Iodoform, and suitable excipients.

Description: Iodocal Paste is a highly antibacterial intracanal dressing indicated for routine use between endodontic treatment visits to disinfect root canals and manage complicated cases requiring extended calcium hydroxide therapy. It may also be used as a temporary root canal disinfectant, for vital and non-vital pulpotomy procedures, and for pulpectomies in deciduous teeth.

Directions for Use: Use under rubber dam isolation and follow standard endodontic procedures. Clean, shape, and dry the root canal. Attach the applicator tip securely to the luer-lock syringe. Gently dispense the paste by applying steady pressure to the syringe plunger. Do not force the plunger. The paste may also be placed into the canal using a lentulo spiral. Remove any excess material with a sterile cotton pellet. Seal the access cavity with the preferred temporary or permanent restorative material. Discard the applicator tip and replace the cap immediately after use. The paste can be easily removed using endodontic files and sodium hypochlorite irrigation.

Warnings: Avoid injecting the paste beyond the root apex. The product is intended for use only by qualified dental professionals. Do not use in patients with known hypersensitivity to any of its components. Avoid prolonged or repeated contact with the skin. Due to its high pH, the paste may cause irritation or burns to the skin and oral mucosa. In case of skin contact, wash immediately with plenty of water and soap. If the product comes into contact with the eyes, rinse thoroughly with water and seek medical attention. Discontinue use if irritation or an allergic reaction occurs.

Precautions for Use: Replace the syringe cap immediately after use. Store the product in a cool, dry place below 25°C. Keep out of the reach of children.

Storage Conditions: Store below 25°C in a dry place. Dispose of the product according to local regulations.

Shelf Life: 3 years from the date of manufacture.

Presentation: 1 × 1.5 g syringe, Applicator Tips—Ref. No.: **ED10-01**, 1 × 3 g syringe, Applicator Tips—Ref. No.: **ED10-02**, 2 × 1.5 g syringe, Applicator Tips—Ref. No.: **ED10-03**, 2 × 3 g syringe, Applicator Tips—Ref. No.: **ED10-04**

NOTE: TO BE USED BY DENTAL PROFESSIONALS ONLY.

Manufactured By :
GOLDEN BIRD INTERNATIONAL
A-4, Vishwakarma Industrial Estate,
Baghpat Road, Meerut-250002-India
info@bindiagri.com | www.dentakris.com
LIC.No: MFG/MD/2024/000341
®-Registered Trademark
Customer Care No. +91-7905171053

☞ Symbol for "BATCH CODE" ☞ Symbol for "USE BY" ☞ Symbol for "NON STERILE" ☞ Symbol for "KEEP DRY" ☞ Symbol for "KEEP AWAY FROM SUNLIGHT" ☞ Symbol for "KEEP OUT OF THE REACH OF CHILDREN" ☞ Symbol for "MANUFACTURER NAME & ADDRESS" ☞ Symbol for "CONSULT INSTRUCTION FOR USE" ☞ Symbol for "CATALOGUE NUMBER" ☞ Symbol for "DO NOT USE IF PACKAGE IS DAMAGE" ☞ Symbol for "TEMPERATURE LIMITATION"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding.