

KRIS -GEL

Edta , Carbamide Peroxide Gel

TECHNICAL & SAFETY INSTRUCTIONS FOR USE



INTENDED USE- It is particularly helpful during re-processing of inappropriately filled root canals; facilitates widening and mechanical preparation of narrow root canals. It has lubricating properties, that help sliding of tool into the canal, it is easy for application into the canal and penetrates it easily. Reduces the risk of tool fracture in the canal.

COMPOSITION: Urea Peroxide, Polysorbate, EDTA, incipient.

DESCRIPTION: The product contains disodium EDTA, which reacts with mineral components of hard dental tissues (chelating) making them susceptible to mechanical treatment. Moreover, KRIS- gel exhibits lubricating properties due to its texture. Colouration of the product makes it well visible in the pulp chamber - exposition of canal orifice; - chemical widening and restoration of patency of root canals; - removal of greasy material from canal walls; - restoration of canal patency in the presence of dentinomas

DIRECTIONS FOR USE: Remove the cap from the syringe and mount disposable tip (blunt disposable needle) on place; 1. Using the disposable tip, apply small amount of the product in the pulp chamber or directly on the canal tool, and then prepare the canal mechanically. 2. During mechanical treatment rinse the canal with solution of sodium hypochlorite or chlorhexidine. 3. After application of the product, move the syringe piston slightly back, to remove the gel from the tip. Close the syringe with the cap. Pay particular attention to thorough washout of the product from the canal, as failure to do it may result in excessive "softening" and discolouration of dentin. 4. Excessive "softening" of dentin, in particular close to anatomical orifice, may lead to root perforation, in particular when tools over No. 30 are used, which are not as flexible as thinner ones and, with softened dentin, may cause perforation. 5. During washout of the product, follow precautions applicable to given rinsing agent. 6. Do not use the product in the case of hypersensitivity to any of its components. 7. Please remember that given amount of KRIS-GEL gel may bind only limited number of calcium ions; thus, the gel applied into the canal should be replaced frequently, to maintain it chelating action. Pay particular attention to firm mounting of the disposable tip. During application of the product do not push the syringe piston too strongly as in may cause detachment of the tip from the syringe.

CONTRA INDICATIONS: Avoid prolonged inhalation. Do not take internally contact with the skin may cause irritation, burns of hypersensitivity. If contact with skin occurs immediately wipe off thoroughly with cotton and alcohol and then wash well with soap and water. If skin rash and sensitization or other allergic reaction occurs discontinue use and seek medical attention. If the product comes into contact with the eyes, rinse thoroughly with fresh water and consult a physician.

Warning: Avoid prolonged or repeated contact with skin (allergic contact dermatitis), and eyes. Avoid prolonged inhalation. Do not take it internally. Contact with the skin may cause irritation, burns of hypersensitivity. If contact with skin occurs immediately wipe off thoroughly with cotton and alcohol and then wash well with soap and water. If skin rash and sensitization or other allergic reaction occurs discontinue use and seek medical attention. If the product comes into contact with the eyes, rinse thoroughly with fresh water and consult a physician. It should not be used with patients who have a history of severe allergic reaction to any of the components.

PRECAUTIONS FOR USE: Due to sensitivity to moisture, close the syringe immediately after use.

Presentation : 1 x 1.5 gm REF NO. ED03-01, 1 x 3 gm REF NO. ED03-02, 1 x 2 x 1.5 gm REF NO. ED03-03, 2 x 3 gm REF NO. ED03-04, 6 x 3 gm REF NO. ED03-05, Syringes with applicator tips.

STORAGE CONDITIONS: The product should be stored below 25°C.

DISPOSAL CONDITIONS: To be disposed off based on the prevailing rules of the country in which it is used.

EXPIRY : 3 years from the date of manufacturing.

NOTE: TO BE USED BY DENTAL PROFESSIONALS ONLY.

Manufactured By:
Golden Bird International
Plot No. A-4 Vishwakarma
Industrial Estate, Village Panchli
Khurd Meerut-250002 India, info@indiagbi.com
www.dentakris.com, GOLDEN-Meeru-UP/MMD/005962
®-Registered Trademark Only

- Symbol for "BATCH CODE" Symbol for "USE BY" Symbol for "NON STERILE" Symbol for "KEEP DRY" Symbol for "CAUTION"
- SYMBOL FOR EUROPEAN COMMUNITY AUTHORIZED REPRESENTATIVE Symbol for "KEEP AWAY FROM SUNLIGHT"
- Symbol for "KEEP OUT OF THE REACH OF CHILDREN" Symbol for "MANUFACTURER NAME & ADDRESS" Symbol for "CONSULT"
- INSTRUCTION FOR USE Symbol for "CATALOGUE NUMBER" Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"
- Symbol for "TEMPERATURE LIMITATION"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding.