

KRIS-TEMP



Temporary Filling Material Eugenol Free TECHNICAL & SAFETY INSTRUCTIONS FOR USE

INTENDED USE- KRIS-TEMP is used for temporary filling of dental cavities. It is a temporary seal for endodontic applications and inlay/onlay preparations. Temp Paste radiopaque provides good retention, easy removal, reduce dentine hyper-sensitivity and post operative pain due to preparations for direct and indirect restorations.

COMPOSITION: Zinc Sulphate, Resin, Zinc Oxide, etc.

DESCRIPTION: KRIS - TEMP provisional filling material is based on powder of zinc-sulphate cement and dough-adjuster with dyes and fragrances. The material is cured with oral cavity moisture during 2 hours. The dough provides tooth cavity hermetization, prevents teeth from coloring when filling with amalgams, it is non-soluble in saliva.

DIRECTIONS FOR USE: KRIS TEMP provisional filling material is introduced into pretreated and dried carious cavity by means of a smoother. The dough hardening time is 2 hours (the patient is recommended not to take food during this time). The dough is easily removed from the cavity by a slight movement of an excavator or a dental explorer. Some dough should remain in the cavity for pulp protection. Then the cavity should be filled with permanent filling material or amalgam.

WARNING: Avoid prolonged inhalation. Do not take internally Contact with the skin may cause irritation, burns of hypersensitivity. If contact with skin occurs immediately wipe off thoroughly with cotton and alcohol and then wash well with soap and water. If skin rash and sensitization or other allergic reaction occurs discontinue use and seek medical attention. If the product comes into contact with the eyes, rinse thoroughly with fresh water and consult a physician.

PRECAUTIONS FOR USE: Due to sensitivity to moisture, close the tube or jar immediately after removing the material, since otherwise the viscosity of the material increases (due to the entry of moisture), and the material may no longer be extractable.

STORAGE CONDITIONS: The product should be stored below 25°C.

DISPOSAL CONDITIONS: To be disposed off based on the prevailing rules of the country in which it is used.

PRESENTATION : 1 x 25 gm RefNo. ED12-01, 1 x 40 gm RefNo. ED12-02, Plastic Jar.

EXPIRY: 3 years from the date of manufacturing.

NOTE: TO BE USED BY DENTAL PROFESSIONALS ONLY.

Manufactured By:
Golden Bird International
Plot No. A-4 Vishwakarma
Industrial Estate, Village Panchli
Khurd Meerut-250002 India, info@indiagbi.com
www.dentakris.com, GOLDEN-Meeru-UP/M/MD/005962
®-Registered Trademark Only

Symbol for "BATCH CODE" Symbol for "USE BY" Symbol for "NON STERILE" Symbol for "KEEP DRY" Symbol for "CAUTION"
 SYMBOL FOR EUROPEAN COMMUNITY AUTHORIZED REPRESENTATIVE Symbol for "KEEP AWAY FROM SUNLIGHT"
 Symbol for "KEEP OUT OF THE REACH OF CHILDREN" Symbol for "MANUFACTURER NAME & ADDRESS" Symbol for "CONSULT
INSTRUCTION FOR USE" Symbol for "CATALOGUE NUMBER" Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"
 Ssymbol for "TEMPERATURE LIMITATION"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding.