

ROOT X

Root Canal Sealer

TECHNICAL & SAFETY INSTRUCTIONS FOR USE



Product Description: Root Canal sealer is an antibacterial, non inflammatory sealer for filling dental root canals. Can tightly seal the apex of the root and the prepared root canal and can be easily removed with Gutta Percha Point when refilling is necessary. Composition based on Grassman formula is radiopaque, non staining non irritation with excellent thermal insulation. Very long working time and excellent lubricating qualities.

Application: Root Canal sealing with and without Gutta Percha.

Root Canal filling for infected root canal treatment.

Composition: Powder - Zinc oxide, Bismuth Subcarbonate, Barium Sulphate, Sodium Borate, Idoform Stabelite resins.

Methods of use: The root canal should be thoroughly dried before filling, over filling of root canals should be avoided. Both glass slabs and spatula should be sterilized and thoroughly dry prior to use for mixing cement. Place 2-3 drops of liquid and about 0.5 gm of powder on the slab. Spatulate small portions of the powder into liquid until all particles of powder are dissolved. Gradually add more portions of powder to the mix and mix thoroughly until a very smooth creamy paste is obtained. Introduce the paste into the prepared root canal with a root explorer. Insert the gutta percha and seal the coronal end with base.

Warning: Avoid prolonged or repeated contact with skin (allergic contact dermatitis) and eyes. Avoid prolonged inhalation. Do not take it internally. Contact with the skin may cause irritation, burns of hypersensitivity. If contact with skin occurs immediately wipe off thoroughly with cotton and alcohol and then wash well with soap and water. If skin rash and sensitization or other allergic reaction occurs discontinue use and seek medical attention. If the product comes into contact with the eyes, rinse thoroughly with fresh water and consult a physician. It should not be used with patients who have a history of severe allergic reaction to any of the components.

Precautions: Do not use root canal sealer with patients that have a history a allergic reaction to iodine compounds. Avoids contact with eyes.

Storage: The Product should be stored in cool and dry environment. Close the container carefully after use. Keep away from Moisture, Shelf Life not affected after opening if stored in cool dry place. If not to be used for longer duration, keep them in refrigerator. Allow material to reach room temperature prior to use.

Presentation : 1 x 15 gm , 1 x 8 ml, RefNo. ED15-03 Glass Bottle, 1 x Mixing Pad, 1 X Spoon.

Shelf Life: 3 Years from the date of Manufacturing.

Disposable conditions: To be disposed off based on the prevailing rules of the country in which it is used.

NOTE: TO BE USED BY DENTAL PROFESSIONALS ONLY.

Manufactured By:
Golden Bird International
Plot No. A-4 Vishwakarma
Industrial Estate, Village Panchli
Khurd Meerut-250002 India, info@indiagbi.com
www.dentakris.com, GOLDEN-Meeru-UP/M/MD/005962
®-Registered Trademark Only

LOT Symbol for "BATCH CODE" **U** Symbol for "USE BY" **NS** Symbol for "NON STERILE" **☞** Symbol for "KEEP DRY" **⚠** Symbol for "CAUTION"
EC/REP SYMBOL FOR EUROPEAN COMMUNITY AUTHORIZED REPRESENTATIVE **☞** Symbol for "KEEP AWAY FROM SUNLIGHT"
☞ Symbol for "KEEP OUT OF THE REACH OF CHILDREN" **■** Symbol for "MANUFACTURER NAME & ADDRESS" **[i]** Symbol for "CONSULT INSTRUCTION FOR USE" **REF** Symbol for "CATALOGUE NUMBER" **⊗** Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"
⚡ Ssymbol for "TEMPERATURE LIMITATION"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding.