

# ZINC OXIDE

Lead and Arsenic Free

## TECHNICAL & SAFETY INSTRUCTIONS FOR USE



**Product Description :** Zinc Oxide Powder reacts with Eugenol to form a sedative dental cement or lining under a non composite resin restoration.

**Composition :** Pure Zinc Oxide

**Indications :** Zinc Oxide is used for temporary cementation in combination with eugenol.

**Instructions for Use :** Mix with Eugenol to form a thick paste and place into a tooth cavity either as a dressing on a painful tooth or as a lining under a non composite resin restoration.

**Precautions :** Avoid contact with eye. Inhibits the polymerization of composite resins when used with Eugenol and should not be used as a dressing prior to composite resin restoration.

**Storage :** Store at temperature between 10°C to 24°C. Close the cap carefully after use. Keep away from moisture. Shelf life not affected after opening if stored in a cool dry place.

**Warning :** Avoid prolonged or repeated contact with skin (allergic contact dermatitis), and eyes. Avoid prolonged inhalation. Do not take internally Contact with the skin may cause irritation, burns of hypersensitivity. If contact with skin occurs immediately wipe off thoroughly with cotton and alcohol and then wash well with soap and water. If skin rash and sensitization or other allergic reaction occurs discontinue use and seek medical attention. If the product comes into contact with the eyes, rinse thoroughly with fresh water and consult a physician. It should not be used with patients who have a history of severe allergic reaction to any of the components.

**Disposal Conditions :** To be disposed off based on the prevailing rules of the country in which it is used.

**Presentation :** 1 x 100 gm Plastic Jar, RefNo. ED14-01

**Shelf Life :** 3 years from the date of manufacturing.

**NOTE: TO BE USED BY DENTAL PROFESSIONAL ONLY**

Manufactured By:  
**Golden Bird International**  
Plot No. A-4 Vishwakarma  
Industrial Estate, Village Panchli  
Khurd Meerut-250002 India, info@indiagbi.com  
www.dentakris.com, GOLDEN-Meeru-UP/M/MD/005962  
®-Registered Trademark Only

Symbol for "BATCH CODE" Symbol for "USE BY" Symbol for "NON STERILE" Symbol for "KEEP DRY" Symbol for "CAUTION"  
 SYMBOL FOR EUROPEAN COMMUNITY AUTHORIZED REPRESENTATIVE Symbol for "KEEP AWAY FROM SUNLIGHT"  
 Symbol for "KEEP OUT OF THE REACH OF CHILDREN" Symbol for "MANUFACTURER NAME & ADDRESS" Symbol for "CONSULT INSTRUCTION FOR USE" Symbol for "CATALOGUE NUMBER" Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"  
 Symbol for "TEMPERATURE LIMITATION"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding.