

# KRIS CAL



## Calcium Hydroxide Powder For Cavity Base TECHNICAL & SAFETY INSTRUCTIONS FOR USE

**Intended Use:** Kris Cal Calcium Hydroxide Powder when mixed with deionized water becomes a smooth paste to be used as a dressing for apexification procedures, direct & indirect pulp capping and as a sub base under restoration.

**Composition:** Calcium Hydroxide, calcium phosphate, etc

**Areas of Application:**

- Direct & indirect pulp capping
- Sub base under restoration
- Dressing for apexification procedures
- Thermal Insulation

**Procedure to use Material:** Mix with deionized water to form homogenous paste and place into tooth cavity.

**Storage:** Keep Away from Direct Sunlight, Keep Away from Moisture, Always keep the Bottles and packets tightly closed once opened. Store at Temp. between 5°C to 30°C Shelf life not affected after opening if stored in a cool dry place, If not to be used for a longer duration, please keep them in refrigerator. Allow material to reach room temperature prior to use.

**Disposal Conditions:** To be disposed off based on the prevailing rules of the country in which it is used.

**Warning:** Avoid prolonged or repeated contact with skin (allergic contact dermatitis), and eyes. Avoid prolonged inhalation. Do not take internally. **Eye contact:** if the product comes in contact with eyes it may cause irritation. In case of contact with eyes, rinse immediately with plenty of water and seek medical attention. **Skin Contact:** if comes in contact with Skin, it may cause skin sensitization, If contact with skin occurs immediately wipe off thoroughly with cotton and alcohol and then wash well with soap and water. If skin rash and sensitization or other allergic reaction occurs discontinue use and seek medical attention. It should not be used with patients who have a history of severe allergic reaction to any of the components.

**Presentation :** 1 x 15 gm Ref No. ED08-01

**Shelf Life:** 3 years from date of manufacturing.

### NOTE: TO BE USED BY DENTAL PROFESSIONALS ONLY.

Manufactured By:  
**Golden Bird International**  
Plot No. A-4 Vishwakarma  
Industrial Estate, Village Panchli  
Khurd Meerut-250002 India, info@indiagbi.com  
www.dentakris.com, GOLDEN-Meeru-UP/M/MD/005962  
®-Registered Trademark Only

- Symbol for "BATCH CODE" Symbol for "USE BY" Symbol for "NON STERILE" Symbol for "KEEP DRY" Symbol for "CAUTION"  
 SYMBOL FOR EUROPEAN COMMUNITY AUTHORIZED REPRESENTATIVE Symbol for "KEEP AWAY FROM SUNLIGHT"  
 Symbol for "KEEP OUT OF THE REACH OF CHILDREN" Symbol for "MANUFACTURER NAME & ADDRESS" Symbol for "CONSULT INSTRUCTION FOR USE" Symbol for "CATALOGUE NUMBER" Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"  
 Symbol for "TEMPERATURE LIMITATION"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding.