

ALVO +

Alveoles Dry Socket Paste

TECHNICAL & SAFETY INSTRUCTIONS FOR USE



INTENDED USE: It is used as an anaesthetic and antiseptic alveolar compress after dental extraction. Alvo + provides a quick bleeding stoppage due to presence of Amino Capronic acid in the formulation.

COMPOSITION: Amino-Capronic acid, Centrium Bromide, Glycerine, Polysaachride, Idoform.

DESCRIPTION: Introduced into the socket Alvo + easily adheres to the alveoles due to its fibrous consistency. Assisted by the patient's tongue movements it gradually self eliminates without any intervention by the practitioner or the patient as the healing of the wound takes place.

DIRECTIONS FOR USE: Sterilise the socket with warm normal saline solution. Then, using a pair of sterilized tweezers, apply a sufficient quantity of ALVO +(about 0.20 g) to adequately cover the bottom of the socket and pack gently into place. Do not suture.

CONTRA INDICATIONS: Routine use of the product as a post-extraction dressing is Alvo+ is not to be used on patients having a history of allergy reactions to procaine type anaesthetics or sensitivity to iodine compounds.

WARNING : Avoid prolonged or repeated contact with skin (allergic contact dermatitis), and eyes. Avoid prolonged inhalation. Do not take it internally. Contact with the skin may cause irritation, burns of hypersensitivity. If contact with skin occurs immediately wipe off thoroughly with cotton and alcohol and then wash well with soap and water. If skin rash and sensitization or other allergic reaction occurs discontinue use and seek medical attention. If the product comes into contact with the eyes, rinse thoroughly with fresh water and consult a physician.

PRECAUTIONS FOR USE: Always replace cap immediately after use, keep jar in a dry place below 25° C.

STORAGE CONDITIONS: The product should be stored below 25°C.

DISPOSAL CONDITIONS: To be disposed off based on the prevailing rules of the country in which it is used.

PRESENTATION : 1 x 12 gm RefNo. RD01-01 Jar Container of paste.

EXPIRY: 3 years from the date of manufacturing.

NOTE: TO BE USED BY DENTAL PROFESSIONALS ONLY.

Manufactured By:
Golden Bird International
Plot No. A-4 Vishwakarma
Industrial Estate, Village Panchli
Khurd Meerut-250002 India, info@indiagbi.com
www.dentakris.com. GOLDEN-Meeru-UP/M/MD/005962
®-Registered Trademark Only

Symbol for "BATCH CODE" Symbol for "USE BY" Symbol for "NON STERILE" Symbol for "KEEP DRY" Symbol for "CAUTION"
 SYMBOL FOR EUROPEAN COMMUNITY AUTHORIZED REPRESENTATIVE Symbol for "KEEP AWAY FROM SUNLIGHT"
 Symbol for "KEEP OUT OF THE REACH OF CHILDREN" Symbol for "MANUFACTURER NAME & ADDRESS" Symbol for "CONSULT INSTRUCTION FOR USE" Symbol for "CATALOGUE NUMBER" Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"
 Symbol for "TEMPERATURE LIMITATION"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding.