

# CAVIT LINER

## Cavity Varnish With Fluoride

### TECHNICAL & SAFETY INSTRUCTIONS FOR USE



**Intended Use:** Natural resins, used as varnishing agent of cavity surface in RCT.

**Composition:** Rosin, solvents, etc.

**Areas of Application:** A) It is Ideal cavity liner and dental tubules sealer.

B) As sure perfect protection to exposed dentine.

C) Intermediary varnish prior to filling and gams, cements.

**Procedure to use material:** A small cotton pellet held by cotton forcep is dipped into the varnish just enough to wet the pellet. The cavity varnish is applied to the pulpal area, walls of the cavity preparation, and on to the edge of the margins of the preparation. Any excess varnish can be removed from the enamel with a fresh cotton pellet. A second application of cavity varnish is placed over the first to thoroughly coat the surfaces of the dentin and fill any voids from bubbles created when the first application dries Cavity Liner leaving the resin as a this over the preparation and now a mal gam restoration can be placed.

Cavity varnish should not be used with composites since it retards the curing mechanism.

**Caution:** A) Incase of contact with oral tissue or skin, remove immediately with a sponge or cotton soaked in alcohol and flush with water.

B) In case of contact with eyes, flush immediately with water and seek medical attention.

**Storage Conditions:** The product should be stored below 25°C.

**Disposal Conditions:** To be disposed off based on the prevailing rules of the country in which it is used.

**Warning:** Avoid prolonged or repeated contact with skin (allergic contact dermatitis), and eyes. Avoid prolonged inhalation. Do not take it internally. Contact with the skin may cause irritation, burns of hypersensitivity. If contact with skin occurs immediately wipe off thoroughly with cotton and alcohol and then wash well with soap and water. If skin rash and sensitization or other allergic reaction occurs discontinue use and seek medical attention. If the product comes into contact with the eyes, rinse thoroughly with fresh water and consult a physician. It should not be used with patients who have a history of severe allergic reaction to any of the components.

**Presentation :** 1 x 15 ml RefNo. RD06-01

**Shelf Life:** 3 years from the date of manufacturing.

**NOTE: TO BE USED BY DENTAL PROFESSIONALS ONLY.**

Manufactured By:  
**Golden Bird International**  
Plot No. A-4 Vishwakarma  
Industrial Estate, Village Panchli  
Khurd Meerut-250002 India, info@indiagbi.com  
www.dentakris.com. GOLDEN-Meeru-UP/M/MD/005962  
®-Registered Trademark Only

Symbol for "BATCH CODE" Symbol for "USE BY" Symbol for "NON STERILE" Symbol for "KEEP DRY" Symbol for "CAUTION"  
 SYMBOL FOR EUROPEAN COMMUNITY AUTHORIZED REPRESENTATIVE Symbol for "KEEP AWAY FROM SUNLIGHT"  
 Symbol for "KEEP OUT OF THE REACH OF CHILDREN" Symbol for "MANUFACTURER NAME & ADDRESS" Symbol for "CONSULT INSTRUCTION FOR USE" Symbol for "CATALOGUE NUMBER" Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"  
 Symbol for "TEMPERATURE LIMITATION"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding.