

CHX-2%



Chlorhexidine Gluconate Solution I.P.

TECHNICAL & SAFETY INSTRUCTIONS FOR USE

INTENDED USE:

CHX-2% is a chlorhexidine digluconate solution. It is intended for the irrigation and debridement of root canals after instrumentation in endodontic treatment. Remove smear layer with appropriate irrigant such as EDTA solution or Citric Acid 12.5%.

Instruction for use:

1. Use EDTA 17% as a pre-rinse in the canal to remove smear layer.
2. Rinse with Sodium Hypochlorite (NaOCl).
3. Rinse out the NaOCl with sterile water or anesthetic solution
4. Fill syringe with CHX-2%(5-10 ml recommended). Attach irrigation tip securely to syringe.
5. Continuously irrigate for a minimum of 2 minutes. (10 – 20 ml per canal recommended).
6. Remove and dispose of syringe and irrigating tip.
7. Dry and obturate the root canal.

Precautions:

1. Read MSDS for material handling and safety information.
2. Do not ingest, do not inject.
3. Not to be used as an oral rinse

Warnings:

1. Products containing chlorhexidine must not be mixed with products containing sodium hypochlorite to prevent formation of a reddish-brown low solubility precipitate.
2. Keep out of the reach of children.
3. Avoid contact with skin and eyes.

Storage: Store at room temperature, dry area, away from heat and direct sunlight.

Presentation: 1 x 100 ml RefNo. ED18-01, 1 x 500 ml RefNo. ED18-02

Shelf Life: 3 years from date of manufacturing.

NOTE: TO BE USED BY DENTAL PROFESSIONAL ONLY

Manufactured By:
Golden Bird International
Plot No. A-4 Vishwakarma
Industrial Estate, Village Panchli
Khurd Meerut-250002 India, info@indiagbi.com
www.dentakris.com, GOLDEN-Meeru-UP/M/MD/005962
®-Registered Trademark Only

Symbol for "BATCH CODE" Symbol for "USE BY" Symbol for "NON STERILE" Symbol for "KEEP DRY" Symbol for "CAUTION"
 SYMBOL FOR EUROPEAN COMMUNITY AUTHORIZED REPRESENTATIVE Symbol for "KEEP AWAY FROM SUNLIGHT"
 Symbol for "KEEP OUT OF THE REACH OF CHILDREN" Symbol for "MANUFACTURER NAME & ADDRESS" Symbol for "CONSULT INSTRUCTION FOR USE" Symbol for "CATALOGUE NUMBER" Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"
 Symbol for "TEMPERATURE LIMITATION"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding.