

# D-NERVE



## Arsenicless Dough For Pulp Devitalization TECHNICAL & SAFETY INSTRUCTIONS FOR USE

**INTENDED USE-** D-NERVE PASTE is used for pulp devitalisation in treatment of pulpitis or its amputation. The dough is also used for treatment of residual root pulpitis of provisional and permanent teeth.

**COMPOSITION:** Para formaldehyde, Lidocaine Base, Phenol, incipient.

**DESCRIPTION:** Painless pulp devitalisation of permanent teeth due to the presence of lidocaine. Contain par formaldehyde to enable rapid mummification of the pulp. With phenol for its antiseptic properties, intensifies the hardening effect of par formaldehyde and increases the antiseptic effect of the preparation.

**DIRECTIONS FOR USE:** Remove the cap from the syringe and mount disposable tip (blunt disposable needle) on place; Apply aseptically a pellet (about 10 mg of paste) to the pulp, after having removed the carious dentine and exposed the pulp. Seal with a cement and open again after 7 to 10 days. Then, if necessary, repeat the treatment. After extirpation of the pulp and treatment of the canals, the tooth can be filled permanently. The product is to be used only twice at the most, at the above mentioned dose, in the treatment of any one tooth.

**WARNING:** Avoid prolonged inhalation. Do not take internally contact with the skin may cause irritation, burns of hypersensitivity. If contact with skin occurs immediately wipe off thoroughly with cotton and alcohol and then wash well with soap and water. If skin rash and sensitization or other allergic reaction occurs discontinue use and seek medical attention. If the product comes into contact with the eyes, rinse thoroughly with fresh water and consult a physician.

**PRECAUTIONS FOR USE:** Due to sensitivity to moisture, close the syringe immediately after use.

**STORAGE CONDITIONS:** The product should be stored below 25°C.

**DISPOSAL CONDITIONS:** To be disposed off based on the prevailing rules of the country in which it is used.

**PRESENTATION :** 1 x 1.5 gm, RefNo. RD07-01, 1 x 3 gm, RefNo.RD07-02, 2 x 1.5 gm, RefNo. RD07-03, 2 x 3 gm, RefNo. RD07-04, Applicator Tips.

**SHELF LIFE:** 3 years from the date of manufacturing.

**NOTE: TO BE USED BY DENTAL PROFESSIONALS ONLY.**

Manufactured By:  
**Golden Bird International**  
Plot No, A-4 Vishwakarma  
Industrial Estate, Village Panchli  
Khurd Meerut-250002 India, info@indiagbi.com  
www.dentakris.com, GOLDEN-Meeru-UP/M/MD/005962  
®-Registered Trademark Only

Symbol for "BATCH CODE" Symbol for "USE BY" Symbol for "NON STERILE" Symbol for "KEEP DRY" Symbol for "CAUTION"  
 SYMBOL FOR EUROPEAN COMMUNITY AUTHORIZED REPRESENTATIVE Symbol for "KEEP AWAY FROM SUNLIGHT"  
 Symbol for "KEEP OUT OF THE REACH OF CHILDREN" Symbol for "MANUFACTURER NAME & ADDRESS" Symbol for "CONSULT INSTRUCTION FOR USE" Symbol for "CATALOGUE NUMBER" Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"  
 Symbol for "TEMPERATURE LIMITATION"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding.