

ENDO X



Root Canal Obturation Cement

TECHNICAL & SAFETY INSTRUCTIONS FOR USE

Product Description: Endo X is a radiopaque, non-resorbable Zinc Oxide-Eugenol based root canal sealer. Its composition is well tolerated by the tissues and provides antiseptic, anti inflammatory and germicidal action. Excellent anti inflammatory properties due to the presence of Hydrocortisone Acetate.

Application: Permanent Root canal filling and sealing in association with gutta percha point.

Composition:

Powder contains: Zinc Oxide, Hydrocortisone Acetate, Barium Sulphate, Thymol Iodide, Magnesium Stearate.

Liquid contains: Eugenol & Exceipients.

Method of Use: Mix carefully for approximately 45-60 seconds, one scoop of powder and five drops of liquid to obtain a thick mix, which can easily be introduced into the canal with suitable root canal filler, or in conjunction with a silver, acrylic or gutta percha point. Net setting time is 45-60 minutes from start of mix.

Disposal conditions: To be disposed off based on the prevailing rules of the country in which it is used.

Disclaimer: The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instruction of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding.

Warning: Avoid prolonged or repeated contact with skin (allergic contact dermatitis), and eyes. Avoid prolonged inhalation. Do not take it internally. Contact with the skin may cause irritation, burns of hypersensitivity. If contact with skin occurs immediately wipe off thoroughly with cotton and alcohol and then wash well with soap and water. If skin rash and sensitization or other allergic reaction occurs discontinue use and seek medical attention. If the product comes into contact with the eyes, rinse thoroughly with fresh water and consult a physician. It should not be used with patients who have a history of severe allergic reaction to any of the components.

Storage: Store at room temperature between 5~ 35°C. Protect from direct sunlight. Close the container carefully after use. Keep away from moisture. Shelf life not affected after opening if stored in a cool & dry place.

Presentation : 1 x 15 gm, 1 x 10 ml, RefNo. RD16-03, 1 x Spoon, 1 x Mixing Pad.

Shelf life: 3 years from date of manufacturing.

NOTE: TO BE USED BY DENTAL PROFESSIONAL ONLY

Manufactured By:
Golden Bird International
Plot No. A-4 Vishwakarma
Industrial Estate, Village Panchli
Khurd Meerut-250002 India, info@indiagbi.com
www.dentakris.com, GOLDEN-Meeru-UP/M/MD/005962
®-Registered Trademark Only

Symbol for "BATCH CODE" Symbol for "USE BY" Symbol for "NON STERILE" Symbol for "KEEP DRY" Symbol for "CAUTION"
 SYMBOL FOR EUROPEAN COMMUNITY AUTHORIZED REPRESENTATIVE Symbol for "KEEP AWAY FROM SUNLIGHT"
 Symbol for "KEEP OUT OF THE REACH OF CHILDREN" Symbol for "MANUFACTURER NAME & ADDRESS" Symbol for "CONSULT INSTRUCTION FOR USE" Symbol for "CATALOGUE NUMBER" Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"
 Symbol for "TEMPERATURE LIMITATION"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding.