

GI-LC



Light Cured Glass Ionomer Cement

TECHNICAL & SAFETY INSTRUCTIONS FOR USE

Product Description : GI-LC single component, light cured glass ionomer cement for lining, extended fissure sealing and smaller lesions.

Composition : Urethane Dimethacrylate resin, Bisphenol A Glycidyle Methacrylate, Barium Sulphate, FluoroAluminoSilicate Glass Powder, Silanated Barium Glass Powder, Amorphous Fumed Silica, Photoinitiators and Stabilizers.

Indications : ♦ Lining under all kinds of filling materials ♦ Extended fissure sealing ♦ Restoration of smaller lesions.

Instructions for use : Thoroughly clean and air dry the cavity. Apply material to the cavity directly with the help of applicator tip. Do not exceed the thickness of 1mm. Light cure for 30- 40 sec using standard curing light and proceed with standard restoration technique. Warning : Avoid prolonged or repeated contact with skin (allergic contact dermatitis), and eyes. Avoid prolonged inhalation. Do not take internally contact with the skin may cause irritation, burns of hypersensitivity. If contact with skin occurs immediately wipe off thoroughly with cotton and alcohol and then wash well with soap and water. If skin rash and sensitization or other allergic reaction occurs discontinue use and seek medical attention. If the product comes into contact with the eyes, rinse thoroughly with fresh water and consult a physician. It should not be used with patients who have a history of severe allergic reaction to any of the components.

Contraindications : GI-LC is contraindicated for use in patients with known allergies to methacrylate or any other if its components.

Storage Conditions : Store at temperature between 10°C to 24°C. Close the cap carefully after use. Keep away from moisture. Shelf life not affected after opening if stored in a cool dry place.

Disposal conditions : To be disposed off based on the prevailing rules of the country in which it is used.

Warning: Avoid prolonged or repeated contact with skin (allergic contact dermatitis), and eyes. Avoid prolonged inhalation. Do not take it internally. Contact with the skin may cause irritation, burns of hypersensitivity. If contact with skin occurs immediately wipe off thoroughly with cotton and alcohol and then wash well with soap and water. If skin rash and sensitization or other allergic reaction occurs discontinue use and seek medical attention. If the product comes into contact with the eyes, rinse thoroughly with fresh water and consult a physician. It should not be used with patients who have a history of severe allergic reaction to any of the components.

Presentation : 1 x 1.5 gm, RefNo. RD17-01, 1 x 2 gm, RefNo. RD17-02, Applicator Tip.

Shelf Life : 3 years from the date of manufacturing.

NOTE: TO BE USED BY DENTAL PROFESSIONAL ONLY

Manufactured By:
Golden Bird International
Plot No. A-4 Vishwakarma
Industrial Estate, Village Panchli
Khurd Meerut-250002 India, info@indiagbi.com
www.dentakris.com, GOLDEN-Meeru-UP/M/MD/005962
®-Registered Trademark Only

Symbol for "BATCH CODE" Symbol for "USE BY" Symbol for "NON STERILE" Symbol for "KEEP DRY" Symbol for "CAUTION"
 SYMBOL FOR EUROPEAN COMMUNITY AUTHORIZED REPRESENTATIVE Symbol for "KEEP AWAY FROM SUNLIGHT"
 Symbol for "KEEP OUT OF THE REACH OF CHILDREN" Symbol for "MANUFACTURER NAME & ADDRESS" Symbol for "CONSULT INSTRUCTION FOR USE" Symbol for "CATALOGUE NUMBER" Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"
 Symbol for "TEMPERATURE LIMITATION"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding.