

KRIS BOND+

Light-Curing Self Etch Bonding Agent



TECHNICAL & SAFETY INSTRUCTIONS FOR USE

Product Description: Kris Bond+ is a 7th-Generation Light-Curing Self-Etch Bonding Agent that combines etching, priming, and bonding in a single adhesive system, eliminating the need for separate etching and priming steps. Kris Bond+ adhesive can be used for the bonding of both direct and indirect restorations.

Composition: HEMA Phosphate, Urethane Dimethacrylate (UDMA), Triethylene Glycol Dimethacrylate (TEGDMA), Bisphenol A Diglycidyl Methacrylate (Bis-GMA), 2-HydroxyEthyl Methacrylate (HEMA), Tertiary Butanol, Photoinitiators and Stabilizer.

Indications: Direct Applications: 1. Light-cured composite and compomer restorations. 2. Composite /ceramic/ metal repairs. 3. Cavity sealing for amalgam restorations. 4. Light-cured or dual-cured core build-ups (always light-cure an incremental Build-up of core material).

Indirect Applications: 1. Veneers 2. Porcelain, composite, and metal-based inlays, onlays, crowns, bridges. 3. Endodontic posts. 4. Cavity sealing as a pretreatment for indirect restorations.

Instructions for use: Bonding Composites/Compomers to Dentin/Enamel Veneers, crowns, bridges, inlays, onlays and metal-based restorations (used in conjunction with a luting cement) 1. Prepare cavity: Wash thoroughly with water spray and air dry (Do not desiccate). Note: For optimal adhesion, always prep (bevel) all enamel and/or margins prior to application of Kris Bond+. 2. Shake Kris Bond+ adhesive bottle briefly. Note: Upon first use, shake the adhesive bottle vigorously for 20-30 seconds. 3. Dispense 2-3 drops of Kris Bond+ adhesive into a clean well. Replace cap immediately after dispensing. 4. Using the disposable applicator brush, apply a generous amount of adhesive to the enamel/dentin surface. Scrub the surface with a brushing motion for 20 seconds. 5. Apply a second application of Kris Bond+ adhesive with a brushing motion for 20 seconds. 6. Dry the adhesive with gentle air first and then medium air for at least 5 seconds with oil-free air. 7. Light cure for 20-30 seconds. 8. Apply composite according to manufacturer's instructions for use.

Instructions for Bonding Indirect Restorations: Follow Steps 1-7 as described above. After light curing the adhesive, apply the resin luting cement according to its Instructions for Use, seat the restoration, remove excess cement, and cure as recommended by the cement manufacturer.

Contraindications: 1. Do not use Kris Bond+ adhesive if a patient is known to be allergic to any of the materials ingredients or if the stipulated working technique cannot be used. 2. Kris Bond+ adhesive must not be used in conjunction with purely self-curing composite materials. Dual curing materials must be light activated. 3. Do not use Kris Bond+ adhesive for direct pulp capping.

Storage Conditions: Store at temperature between 10° to 24°C. Close the cap carefully after use. Keep away from moisture. Shelf life not affected after opening if stored in a cool dry place.

Warning: Kris Bond+ adhesive is an irritant. Contact with the skin, mucous membranes or eyes should be avoided. If case of skin contact, immediately wash with soap and water. If the material should come into contact with the eyes, immediately rinse with copious amounts of water for at least 15 minutes and consult a physician or ophthalmologist. In rare cases, contact with the skin may lead to a sensitization towards methacrylates. Commercial medical gloves may not provide complete protection against methacrylate sensitization.

Disposal Conditions: To be disposed of based on the prevailing rules of the country in which it is used.

Presentation: 1 × 5 ml— Ref. No.: RD24-01, Plastic Dropper Bottles.

Shelf Life: 3 years from the date of manufacturing.

NOTE: TO BE USED BY DENTAL PROFESSIONALS ONLY.

Manufactured By :
GOLDEN BIRD INTERNATIONAL
A-4, Vishwakarma Industrial Estate,
Baghpat Road, Meerut-250002, India
info@indiagbi.com | www.dentakris.com
LIC NO: MFG/MD/2024/000341
®-Registered Trademark
Customer Care No. +91-7905171053

☐ Symbol for "BATCH CODE" ☑ Symbol for "USE BY" ⚠ Symbol for "NON STERILE" ☂ Symbol for "KEEP DRY" ☀ Symbol for "KEEP AWAY FROM SUNLIGHT" 👶 Symbol for "KEEP OUT OF THE REACH OF CHILDREN" 🏢 Symbol for "MANUFACTURER NAME & ADDRESS" 📖 Symbol for "CONSULT INSTRUCTION FOR USE" 📗 Symbol for "CATALOGUE NUMBER" 🚫 Symbol for "DO NOT USE IF PACKAGE IS DAMAGE" 🌡 Symbol for "TEMPERATURE LIMITATION"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding.