

KRIS CLOT +

Haemostatic Gel

TECHNICAL & SAFETY INSTRUCTIONS FOR USE



INTENDED USE- KRIS CLOT + dental material is used as a haemostatic in case of capillary bleeding, for gum retraction when making imprints and treating cervical caries, for prophylaxes purpose and for treating root canals in case of apical bleeding.

COMPOSITION: Ferrous sulphate, Aluminium chloride, centimonium bromide, etc.

DESCRIPTION: KRIS CLOT+. are applied without spreading on mucosa. The main component of KRIS CLOT + materials is aluminum chloride (i.e. haemostatic) which intensifies centimonium bromide featuring bactericidal effect and for quicker homeostasis, materials are not toxic and can be easily used. The materials do not feature irritation effect and are easily washed off. Haemostatic Gel material on Fe basis can change color of gum tissue for 1-2 days.

DIRECTIONS FOR USE: Sterile cotton wad smeared with KRIS CLOT + should be applied under pressure to bleeding surface for 1 minute. In case of strong bleeding, the part of gum to be treated should be isolated with cotton wad. Some gel should be applied for 1 minute. The retraction cord wetted with "Re-cord" liquid should be applied to provide stronger haemostatic. Then gel should be washed off and gum should be dried with airflow. For root canal treatment, gel is introduced into canal by means of paper point or root needle for one minute. Then the canal should be washed with solution of sodium chloride and dried out.

CONTRAINDICATIONS: Kris Clot+ is not to be used on patients having a history of allergy reactions to type sensitivity to Fe, Al compounds.

WARNING : Avoid prolonged inhalation. Do not take internally Contact with the skin may cause irritation, burns of hypersensitivity. If contact with skin occurs immediately wipe off thoroughly with cotton and alcohol and then wash well with soap and water. If skin rash and sensitization or other allergic reaction occurs discontinue use and seek medical attention .If the product comes into contact with the eyes, rinse thoroughly with fresh water and consult a physician.

RECAUTIONS FOR USE: Always replace cap immediately after use, keep syringe in dry place below 25° C.

STORAGE CONDITIONS: The product should be stored below 25°C.

DISPOSAL CONDITIONS: To be disposed off based on the prevailing rules of the country in which it is used.

PRESENTATION : 2 x 1.5 gm RefNo. RD02-03

2 x 3 gm RefNo. RD02-04, Applicator Needles.

SHELF LIFE: 3 years from the date of manufacturing.

NOTE: TO BE USED BY DENTAL PROFESSIONALS ONLY.

Manufactured By:
Golden Bird International
Plot No. A-4 Vishwakarma
Industrial Estate, Village Panchli
Khurd Meerut-250002 India, info@indiagbi.com
www.dentakris.com. GOLDEN-Meeru-UP/M/MD/005962
®-Registered Trademark Only

LOT Symbol for "BATCH CODE" **☼** Symbol for "USE BY" **☼** Symbol for "NON STERILE" **☼** Symbol for "KEEP DRY" **☼** Symbol for "CAUTION"
EC/REP SYMBOL FOR EUROPEAN COMMUNITY AUTHORIZED REPRESENTATIVE **☼** Symbol for "KEEP AWAY FROM SUNLIGHT"
☼ Symbol for "KEEP OUT OF THE REACH OF CHILDREN" **☼** Symbol for "MANUFACTURER NAME & ADDRESS" **☼** Symbol for "CONSULT INSTRUCTION FOR USE" **REF** Symbol for "CATALOGUE NUMBER" **☼** Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"
☼ Ssymbol for "TEMPERATURE LIMITATION"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding.