

KRIS SHIELD

Dentin Desensitizer Liquid

TECHNICAL & SAFETY INSTRUCTIONS FOR USE



Product Description : Kris Shield Dentin Desensitizer is a highly effective, one step, chair-side treatment for dentinal hypersensitivity caused due to cervical erosions and exposed roots. It acts very fast to seal dentinal tubules. No surface layer is formed and is therefore ideal for use under all indirect restorations.

Composition : 2-Hydroxy Ethyl Methacrylate, Sodium Fluoride and Potassium Nitrate, Ethanol.

Indications : Kris Shield Dentin Desensitizer is used for treatment of dentinal hypersensitivity, before and after scaling, under veneers, inlays crowns, onlays, amalgams, composites, before and after tooth bleaching or whitening.

Instructions for Use : Clean the dentine and rinse with water. Dry the surface to be treated. Apply smallest amount of Kris Shield Desensitizer to the dentinal surface in thin layer with a pellet or brush and leave for 30-60 seconds. Dry the surface by applying a stream of compressed air. Repeat the applications two or three times. In case of persisting symptoms, reapply at one week interval.

Precautions : Kris Shield Desensitizer should be used by a dentist. Do not allow the product to come in contact with eyes or skin. Do not breathe vapor. Advise patient to avoid brushing of treated area and consumption of acidic food or drinks for 24 hours.

Storage Conditions : Store at temperature between 10°C to 24°C. Close the cap carefully after use. Keep away from moisture. Shelf life not affected after opening if stored in a cool dry place.

Disposal conditions : To be disposed off based on the prevailing rules of the country in which it is used.

Warning : Avoid prolonged or repeated contact with skin (allergic contact dermatitis), and eyes. Avoid prolonged inhalation. Do not take internally contact with the skin may cause irritation, burns of hypersensitivity. If contact with skin occurs immediately wipe off thoroughly with cotton and alcohol and then wash well with soap and water. If skin rash and sensitization or other allergic reaction occurs discontinue use and seek medical attention. If the product comes into contact with the eyes, rinse thoroughly with fresh water and consult a physician. It should not be used with patients who have a history of severe allergic reaction to any of the components.

Presentation : 1 x 10 ml RefNo. RD09-01, Glass Bottle.

Shelf life : 3 years from the date of manufacturing.

NOTE: TO BE USED BY DENTAL PROFESSIONAL ONLY

Manufactured By:
Golden Bird International
Plot No, A-4 Vishwakarma
Industrial Estate, Village Panchli
Khurd Meerut-250002 India, info@indiagbi.com
www.dentakris.com, GOLDEN-Meeru-UP/M/MD/005962
®-Registered Trademark Only

LOT Symbol for "BATCH CODE" **USE BY** Symbol for "USE BY" **NON STERILE** Symbol for "NON STERILE" **KEEP DRY** Symbol for "KEEP DRY" **CAUTION** Symbol for "CAUTION"
EC REP SYMBOL FOR EUROPEAN COMMUNITY AUTHORIZED REPRESENTATIVE **KEEP AWAY FROM SUNLIGHT** Symbol for "KEEP AWAY FROM SUNLIGHT"
KEEP OUT OF THE REACH OF CHILDREN Symbol for "KEEP OUT OF THE REACH OF CHILDREN" **MANUFACTURER NAME & ADDRESS** Symbol for "MANUFACTURER NAME & ADDRESS" **CONSULT INSTRUCTION FOR USE** Symbol for "CONSULT INSTRUCTION FOR USE" **REF** Symbol for "CATALOGUE NUMBER" **DO NOT USE IF PACKAGE IS DAMAGE** Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"
TEMPERATURE LIMITATION Symbol for "TEMPERATURE LIMITATION"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding.