

PF-X



Self Etch Light Cure Pit and Fissure Sealant TECHNICAL & SAFETY INSTRUCTIONS FOR USE

Product Description : PF-X is a light curing self etch pit and fissure sealant fortified with active hydroxyapatite that has ability to re-mineralize. This nano filled material offers the perfect combination of flow, opacity, and wear resistance. This product is available in three shades(Clear Opaque and Tinted).

Composition : Urethane Dimethacrylate, Triethylene Glycol Dimethacrylate, acidic phosphate monomer, Silanated Barium Glass Powder Amorphous Fumed Silica, coloring reagents, Curing Agents and Stabilizer.

Indications : 1. Sealing/filling of pits and fissures. 2. Sealing/facing of damaged enamel surfaces. 3. Covering of caries predilection sites during orthodontic treatment. 4. Sealing of deciduous teeth.

Method of use : Fissures should be sealed as early as possible, as soon as the eruption of the tooth allows treatment and moisture control is possible. After isolating the operating site, thoroughly clean the surfaces to be sealed (rotating brush/cleaning paste free of fluorides), spray and dry. In case of incipient carious lesions or unclear diagnosis, extend the fissure with a fine instrument, if necessary.

Application : Remove the protective cap from the syringe. Screw a new applicator tip onto the PF-X syringe. PF-X flows homogeneously from the applicator tip upon medium pressure and thus is very easy to dispense accurately. Apply PF-X from the margin into the fissure or cavity and allow it to penetrate for 15 - 20 sec. Remove any air bubbles with the fine applicator tip.

Light-curing : Polymerise PF-X for 40 sec per surface and check the sealing. Conventional polymerisation devices are suitable for light curing this material. The light output polymerisation devices should be above 800 mW/cm². Remove excess material cautiously (e.g. finishing diamond), polish if necessary (rubber polisher).

Caution : Avoid contact of unpolymerized material with skin/mucous membrane or eyes. Unpolymerized PF-X may cause slight irritation and, in rare cases, may lead to a sensitization against methacrylates. Commercial medical gloves do not provide protection against the sensitizing effect of methacrylates.

Storage Conditions : 1. Close PF-X syringe immediately after use. 2. Store material at 10–24°C (50–75°F). 3. Syringes should not be disinfected with oxidizing disinfection agents. 4. Keep Away from Moisture, stores in a cool dry place. If not to be used for a longer duration, keep them in refrigerator. Allow material to reach room temperature prior to use. Keep material out of the reach of children! For use in dentistry only.

Keep material out of the reach of children! For use in dentistry only.

Warning : Avoid prolonged or repeated contact with skin (allergic contact dermatitis), and eyes. Avoid prolonged inhalation. Do not take internally. Contact with the skin may cause irritation, burns of hypersensitivity. If contact with skin occurs immediately wipe off thoroughly with cotton and alcohol and then wash well with soap and water. If skin rash and sensitization or other allergic reaction occurs discontinue use and seek medical attention. If the product comes into contact with the eyes, rinse thoroughly with fresh water and consult a physician. It should not be used with patients who have a history of severe allergic reaction to any of the components.

Disposal conditions : To be disposed off based on the prevailing rules of the country in which it is used.

Presentation : 1 x 1.5 gm RefNo. CI03-01, 1 x 2 gm RefNo. CI03-02, 2 x Prebent Tips

Shelf Life : 3 years from the date of manufacturing.

NOTE: TO BE USED BY DENTAL PROFESSIONALS ONLY.

Manufactured By:
Golden Bird International
Plot No. A-4 Vishwakarma
Industrial Estate, Village Panchli
Khurd Meerut-250002 India, info@indiagbi.com
www.dentakris.com, GOLDEN-Meeru-UP/M/MD/005962
®-Registered Trademark Only

Symbol for "BATCH CODE" Symbol for "USE BY" Symbol for "NON STERILE" Symbol for "KEEP DRY" Symbol for "CAUTION"
 SYMBOL FOR EUROPEAN COMMUNITY AUTHORIZED REPRESENTATIVE Symbol for "KEEP AWAY FROM SUNLIGHT"
 Symbol for "KEEP OUT OF THE REACH OF CHILDREN" Symbol for "MANUFACTURER NAME & ADDRESS" Symbol for "CONSULT INSTRUCTION FOR USE" Symbol for "CATALOGUE NUMBER" Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"
 Symbol for "TEMPERATURE LIMITATION"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding.