

POLY PASTE

Prophylaxis Paste with Fluoride



TECHNICAL & SAFETY INSTRUCTIONS FOR USE

Product Description: POLY PASTE is a fluoride-containing prophylaxis paste formulated with a unique blend of polishing and cleaning agents for professional dental prophylaxis procedures. It contains 1% Sodium Fluoride to provide post-treatment comfort while effectively cleaning and polishing tooth surfaces. The paste is available in a variety of pleasant fruity flavors and attractive colors to enhance the patient's treatment experience. POLY PASTE is intended for professional dental use only.

Intended Use: POLY PASTE is intended for professional cleaning, stain removal, plaque removal, and polishing of natural teeth during routine dental prophylaxis procedures.

Indications for Use: POLY PASTE is indicated for: Professional cleaning and polishing of tooth surfaces. Removal of plaque, pellicle, and extrinsic stains. Polishing of teeth following scaling and prophylaxis procedures. Preparation of tooth surfaces prior to selected restorative and preventive dental procedures..

Composition: Sodium Fluoride (1%), Hydrated Silica (selected particle sizes), aqueous and non-aqueous vehicle, Flavoring Agents, Coloring Agents, Thickening and Stabilizing Agents, Preservatives.

Directions for Use: 1. Thoroughly scale the teeth to remove calculus before polishing. 2. Dispense a sufficient amount of POLY PASTE into a clean dappen dish or other suitable dispensing container. 3. A saliva ejector may be used during the procedure. 4. Load a rubber prophylaxis cup or soft polishing brush with the paste. 5. Before starting the handpiece, apply a small amount of paste onto each tooth surface (Note : gently stir or mix the paste before use until a uniform consistency is achieved). 6. Operate the handpiece at low speed and polish each tooth surface for approximately 10 seconds using light pressure. 7. Ensure the edge of the rubber cup gently extends below the free gingival margin into the gingival sulcus where clinically appropriate. 8. Use fresh paste for each tooth or tooth surface to achieve maximum cleaning efficiency. 9. Continue until all exposed tooth surfaces have been thoroughly cleaned and polished. 10. Rinse thoroughly and instruct the patient to expectorate.

Precautions: 1. For professional dental use only. 2. Perform a complete oral examination and obtain radiographs, where indicated, before prophylaxis treatment. 3. Remove all calculus deposits prior to polishing. 4. Wear appropriate personal protective equipment (gloves, mask, and protective eyewear). 5. Avoid excessive polishing pressure to minimize enamel abrasion. 6. Use a fresh portion of paste for each patient. 7. Close the container immediately after use to prevent contamination and moisture uptake.

Contraindications: Do not use in patients with known hypersensitivity to sodium fluoride or any other component of the product. Do not use on patients with a history of severe allergic reactions to any of the ingredients.

Warnings: Avoid prolonged or repeated contact with skin and eyes. Do not swallow the product. Avoid prolonged inhalation during use. Skin contact may cause irritation or allergic contact dermatitis in susceptible individuals. In case of skin contact, wipe off the material, wash thoroughly with soap and water, and discontinue use if irritation develops. In case of eye contact, rinse immediately with plenty of water and seek medical attention.

Storage: Store at 10°C–24°C in a cool and dry place. Protect from direct sunlight. Keep the container tightly closed after each use. Protect from moisture. The shelf life is not affected after opening, provided the product is properly stored and handled.

Disposal: Dispose of unused material, empty containers, and contaminated accessories according to applicable local regulations for dental or biomedical waste.

Presentation: 1 × 75 g — REF No.: CO02-01; 1 × 200 g — REF No.: CO02-02;

Shelf Life: 3 years from the date of manufacture.

Disposal: Dispose of unused material, empty containers, and contaminated accessories in accordance with local biomedical waste disposal regulations.

NOTE: TO BE USED BY DENTAL PROFESSIONALS ONLY.

Manufactured By :
GOLDEN BIRD INTERNATIONAL
A-4, Vishwakarma Ind. Est.,
Meerut-250002-India
info@indiaebi.com | www.dentakris.com
LIC.No: MFG/MD/2024/000341
®-Registered Trademark
Customer Care No. +91-7905171053

☐ Symbol for "BATCH CODE" ☐ Symbol for "USE BY" ☐ Symbol for "NON STERILE" ☐ Symbol for "KEEP DRY" ☐ Symbol for "KEEP AWAY FROM SUNLIGHT" ☐ Symbol for "KEEP OUT OF THE REACH OF CHILDREN" ☐ Symbol for "MANUFACTURER NAME & ADDRESS" ☐ Symbol for "CONSULT INSTRUCTION FOR USE" ☐ Symbol for "CATALOGUE NUMBER" ☐ Symbol for "DO NOT USE IF PACKAGE IS DAMAGE" ☐ Symbol for "TEMPERATURE LIMITATION"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding.