

POLY PASTE

Prophylaxis Paste With Fluoride

TECHNICAL & SAFETY INSTRUCTIONS FOR USE



Product Description: Poly paste contains a unique blend of polishing and cleaning agents designed for application during standard dental prophylaxis treatment. It contains sodium fluoride at a concentration of 1% fluoridation for post treatment comfort to the patient. Poly Paste is produced in a variety of fruity flavors & amazing colors for pleasant feeling during the treatment procedure. Poly paste is for professional use only.

Application: Poly paste is used for cleaning, plaque removal and polishing of tooth as a part of professionally administered prophylaxis treatment.

Composition: Poly Paste is composed of Sodium Fluoride, Glycerin, Silica in selected and controlled sizes in an aqueous flavored base.

Methods of use: Dispense Poly paste into dappen dish or other suitable container. A saliva ejector may be helpful during the actual prophylaxis. Fill rubber Poly cup with paste. Before starting hand piece, carry paste to field of operation and deposit a small amount on each tooth surface. It is necessary that fresh paste should be provided for each surface in order to obtain maximum cleaning effectiveness. Run rubber cup at slow speed over each tooth surface for approximately 10 seconds. Make sure that the edge of the cup goes below the free margin of the gingival into the gingival sulcus. Continue procedure until all exposed surfaces of the teeth have been thoroughly cleaned and polished. A rubber cup or soft bristle brush may be used on occlusal surfaces.

Important Precautions:

1. All tooth surfaces should be thoroughly scaled to remove calculus.
2. Necessary examinations and radiographs should be taken prior to prophylaxis paste.
3. During use, wear protective glasses and gloves.

Storage: Store at room temperature between 5~ 35 °C. Protect from direct sunlight. Close the container carefully after use. Keep away from moisture. Shelf life not affected after opening if stored in cool & dry place

Disposable conditions: To be disposed off based on the prevailing rules of the country in which it is used.

Disclaimer: The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instruction of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding.

Warning: Avoid prolonged or repeated contact with skin (allergic contact dermatitis) and eyes. Avoid prolonged inhalation. Do not take it internally. Contact with the skin may cause irritation, burns of hypersensitivity. If contact with skin occurs immediately wipe off thoroughly with cotton and alcohol and then wash well with soap and water. If skin rash and sensitization or other allergic reaction occurs discontinue use and seek medical attention. If the product comes into contact with the eyes, rinse thoroughly with fresh water and consult a physician. It should not be used with patients who have a history of severe allergic reaction to any of the components.

Presentation : 1 x 75 gm RefNo. CO02-01

1 x 200 gm RefNo. CO02-02

Shelf Life: 3 years from the date of Manufacturing.

NOTE: TO BE USED BY DENTAL PROFESSIONALS ONLY.



Manufactured By:

Golden Bird International

Plot No. A-4 Vishwakarma
Industrial Estate, Village Panchli
Khurd Meerut-250002 India, info@indiagbi.com
www.dentakris.com, GOLDEN-Meerut-UP/IM/MD/005962
®-Registered Trademark Only

Symbol for "BATCH CODE" Symbol for "USE BY" Symbol for "NON STERILE" Symbol for "KEEP DRY" Symbol for "CAUTION"

SYMBOL FOR EUROPEAN COMMUNITY AUTHORIZED REPRESENTATIVE Symbol for "KEEP AWAY FROM SUNLIGHT"

Symbol for "KEEP OUT OF THE REACH OF CHILDREN" Symbol for "MANUFACTURER NAME & ADDRESS" Symbol for "CONSULT INSTRUCTION FOR USE" Symbol for "CATALOGUE NUMBER" Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"

Symbol for "TEMPERATURE LIMITATION"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding.