

# ZINC+



## Polymer Reinforced Zinc Oxide-Eugenol Cement TECHNICAL & SAFETY INSTRUCTIONS FOR USE

**Product Description:** Zinc+ is a polymer reinforced zinc oxide- Eugenol cement for temporary restorations, materials have excellent physical properties because of polymer resin used as reinforcing agent Zinc+ is quick set, easy to mix with high compressive and tensile strength, excellent abrasion resistance and low erosion.

**Application:** Used as intermediate material (IRM) for class I and II cavities. Temporary restoration in emergency treatment and to seal the coronal portion of the teeth under endodontic treatment. Also used as base under amalgam and nor resin restorations.

**Composition:** Powder - Zinc oxide, Polymer and Liquid Engenol and excipients.

**Methods of use:** Mix Powder and liquid in a ratio of 3 gm to 1ml for about 1 minute to obtains putty consistency. Place the putty into the cavity and remove excess of the product immediately. Allow hardening for 3-4 minutes. Trim the restoration with a tungsten carbide bur.

**Storage:** Store at room temperature between 10- 35 C . Protect from direct sunlight. Close the container carefully after use. Keep away from moisture. Shelf life not affected after opening, if stored in a cool & dry place.

**Disposable conditions:** To be disposed off based on the prevailing rules of the country in which it is used.

**Disclaimer:** The material has been developed solely for professional dental use. Application should Be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instruction of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding.

**Warning:** Avoid prolonged or repeated contact with skin (allergic contact dermatitis) and eyes. Avoid prolonged inhalation. Do not take it internally. Contact with the skin may cause irritation, burns of hypersensitivity. If contact with skin occurs immediately wipe off thoroughly with cotton and alcohol and then wash well with soap and water. If skin rash and sensitization or other allergic reaction occurs discontinue use and seek medical attention. If the product comes into contact with the eyes, rinse thoroughly with fresh water and consult a physician. It should not be used with patients who have a history of severe allergic reaction to any of the components.

**Presentation :** 1 x 15 gm, 1 x 8 ml, Ref No. RD15-03, 1 x spoon, 1 Mixing Pad.

**Shelf Life:** 3 years from the date of Manufacturing.

**NOTE: TO BE USED BY DENTAL PROFESSIONALS ONLY.**

Manufactured By:  
**Golden Bird International**  
Plot No, A-4 Vishwakarma  
Industrial Estate, Village Panchil  
Khurd Meerut-250002 India, info@indiagbi.com  
www.dentakris.com, GOLDEN-Meeru-UP/M/MD/005962  
®-Registered Trademark Only

Symbol for "BATCH CODE" Symbol for "USE BY" Symbol for "NON STERILE" Symbol for "KEEP DRY" Symbol for "CAUTION"  
 SYMBOL FOR EUROPEAN COMMUNITY AUTHORIZED REPRESENTATIVE Symbol for "KEEP AWAY FROM SUNLIGHT"  
 Symbol for "KEEP OUT OF THE REACH OF CHILDREN" Symbol for "MANUFACTURER NAME & ADDRESS" Symbol for "CONSULT INSTRUCTION FOR USE" Symbol for "CATALOGUE NUMBER" Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"  
 Ssymbol for "TEMPERATURE LIMITATION"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding.