

Fluoride Progel

1.23% APF Topical Gel

TECHNICAL & SAFETY INSTRUCTIONS FOR USE



Product Description: Acidulated Fluoride Gel is a topical fluoride treatment containing fluoride ions in an acidulated phosphate base. It is intended to help reduce dental caries and promote enamel remineralization when applied by a dental professional.

Composition: Sodium Fluoride (equivalent to 1.23% fluoride ion), Phosphate Buffer System, Humectants, Thickener, Flavour.

Indications for Use: 1. Prevention of dental caries. 2. Remineralization of early enamel lesions. 3. Management of patients at moderate to high risk of dental caries. 4. Reduction of tooth sensitivity associated with enamel demineralization.

Contraindications: 1. Patients with known hypersensitivity to any component of the product. 2. Patients who are unable to control swallowing during treatment. 3. Not intended for use in patients for whom fluoride therapy is contraindicated.

Warnings: Do not swallow the gel. Avoid contact with eyes and soft tissues. Excessive ingestion may result in fluoride toxicity.

Precautions: 1. Use appropriate suction and isolation during application. 2. Monitor pediatric patients carefully to minimize swallowing. 3. Use only as directed by a dental professional.

Directions for Use: 1. Clean and dry the teeth before application. 2. Fill the fluoride tray with an adequate amount of APF Gel. 3. Position the tray in the patient's mouth and instruct the patient not to swallow. 4. Leave the tray in place for 4 minutes or as recommended by the dental professional. 5. Remove the tray and ask the patient to expectorate thoroughly. 6. Instruct the patient not to eat, drink, or rinse for at least 30 minutes after treatment or as suggested by the dental professional.

Storage Conditions: Store at temperature ranging from 5°C to 30°C. Protect from direct sunlight and moisture. Keep the container tightly closed when not in use.

Presentation: 1 × 450 ml — Ref. No.: CI02-01

Shelf Life: 3 Years from the date of Manufacturing.

NOTE: TO BE USED BY DENTAL PROFESSIONALS ONLY.

Manufactured By :
GOLDEN BIRD INTERNATIONAL
A-4, Vishwakarma Ind. Est.,
Meerut-250002-India
info@indiasgbi.com | www.dentakris.com
LIC NO: MFG/MD/2024/000341
®-Registered Trademark
Customer Care No. +91-7905171053

Symbol for "BATCH CODE" Symbol for "USE BY" Symbol for "NON STERILE" Symbol for "KEEP DRY" Symbol for "KEEP AWAY FROM SUNLIGHT" Symbol for "KEEP OUT OF THE REACH OF CHILDREN" Symbol for "MANUFACTURER NAME & ADDRESS" Symbol for "CONSULT INSTRUCTION FOR USE" Symbol for "CATALOGUE NUMBER" Symbol for "DO NOT USE IF PACKAGE IS DAMAGE" Symbol for "TEMPERATURE LIMITATION"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding.