

Kris Bond

Light Cure Orthodontic Adhesive

TECHNICAL & SAFETY INSTRUCTIONS FOR USE



Product Description : Kris O Bond Primer is a light-cured single-component orthodontic primer that combines etching, priming and Bonding in one bottle. It is an solvent free dental adhesive containing 10-Methacryloyloxydecyl Dihydrogen Phosphate (10-MDP) a functional monomer which helps in Bonding to dentin and enamel.

Composition : BisGMA, Triethylene Glycol Dimethacrylate, Camphorquinone, Dimethyl Amino Ethyl Methacrylate, Ethyl 4-dimethyl Amino Benzoate, 10-Methacryloyloxydecyl Dihydrogen Phosphate, Butylated Hydroxytoluene.

Indications for use : Kris O Bond is indicated for use as a combined etchant and primer in orthodontic treatment and is used only with light curing direct Bonding orthodontic adhesive.

Contraindications : Patients with a history of hypersensitivity to methacrylate monomers.

Precautions : 1. Safety Precautions : The material contains certain methacrylates and other ingredients which may cause irritation or sensitivity in some individuals. If contact occurs with skin, immediately wash with soap and water. Improper use may result in allergic reaction or skin and tissue irritation. If this occurs, discontinue use of this product. Protective gloves and eye wear should be worn while handling this product. If contact occurs with eye. Flush immediately with large volumes of water and seek medical attention immediately. Do not expose materials to elevated temperatures or intense light.

2. Material Precautions : Kris O Bond is a light-curable material. Use it Immediately once it is dispensed in the mixing well. Prolonged exposure to air and light may lead to solvent evaporation and thickening of the adhesive.

Instruction for use : Self-Etching Technique : Wash thoroughly with water spray. Use an absorbent pellet or high volume evacuation (HVE) for 1-2 seconds, to remove excess water (do not desiccate). **a.** Dispense 1-2 drops of Kris O Bond into a clean well. Replace the cap onto the bottle promptly to avoid evaporation of solvent. **b.** Apply two separate coats of Kris O Bond scrubbing the preparation with a microbrush for 10-15 seconds per coat. **c.** Air-drying using an air syringe for at least 10 seconds till there should be no visible movement of the adhesive. The surface should have a uniform glossy appearance; otherwise, apply an additional coat of Kris O Bond and repeat Step 2 and 3. **d.** Light cure for 20 seconds. **e.** Continue with placement of the orthodontic adhesive restorative material according to the manufacturer's instructions.

Selective-Etching Technique : Wash thoroughly with water spray. Etch enamel using 37% phosphoric acid etchant for 15 seconds. Rinse thoroughly. Remove excess water by blotting the surface with an absorbent pellet or high volume evacuation (HVE) for 1-2 seconds, leaving the preparation visibly moist. **a.** Dispense 1-2 drops of Kris O Bond into a clean well. Replace the cap onto the bottle promptly to avoid evaporation of solvent. **b.** Apply two separate coats of Kris O Bond scrubbing the preparation with a microbrush for 10-15 seconds per coat. **c.** Evaporate excess solvent by thoroughly air-drying with an air syringe for at least 10 seconds, there should be no visible movement of the adhesive. The surface should have a uniform glossy appearance; otherwise, apply an additional coat of Kris O Bond and repeat Step 2 and 3. **d.** Light cure for 20 seconds. **e.** Continue with placement of the orthodontic adhesive restorative material according to the manufacturer's instructions.

Storage conditions : Store at temperature between 10°C to 24°C. Close the cap carefully after use. Keep away from moisture. Shelf life not affected after opening if stores in a cool dry place.

Disposal conditions : To be disposed off based on the prevailing rules of the country in which it is used.

Warning: Avoid prolonged or repeated contact with skin (allergic contact dermatitis), and eyes. Avoid prolonged inhalation. Do not take it internally. Contact with the skin may cause irritation, burns of hypersensitivity. If contact with skin occurs immediately wipe off thoroughly with cotton and alcohol and then wash well with soap and water. If skin rash and sensitization or other allergic reaction occurs discontinue use and seek medical attention. If the product comes into contact with the eyes, rinse thoroughly with fresh water and consult a physician. It should not be used with patients who have a history of severe allergic reaction to any of the components.

Presentation : 1 x 5 ml Ref No. OD02-01, Plastic Bottle.

Shelf Life : 3 Years from the date of manufacturing.

NOTE: TO BE USED BY DENTAL PROFESSIONAL ONLY

Manufactured By:
Golden Bird International
 Plot No. A-4 Vishwakarma
 Industrial Estate Village Panchli
 Khurd Meerut-250002 India, info@indiagbi.com
 www.dentalkris.com, GOLDEN-Meeru-UP/PM/MD/005962
 ®-Registered Trademark Only

LOT Symbol for "BATCH CODE" ■ Symbol for "USE BY" ⚠ Symbol for "NON STERILE" ☀ Symbol for "KEEP DRY" ⚠ Symbol for "CAUTION"
 ECEP SYMBOL FOR EUROPEAN COMMUNITY AUTHORIZED REPRESENTATIVE ☀ Symbol for "KEEP AWAY FROM SUNLIGHT"
 ⚠ Symbol for "KEEP OUT OF THE REACH OF CHILDREN" ■ Symbol for "MANUFACTURER NAME & ADDRESS" [i] Symbol for "CONSULT
 INSTRUCTION FOR USE [REF] Symbol for "CATALOGUE NUMBER" ⚠ Symbol for "DO NOT USE IF PACKAGE IS DAMAGED"
 ⚠ Symbol for "TEMPERATURE LIMITATION"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding.